MANUAL DE USO

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Do not drop, mishandle, or expose to temperature or humidity extremes <outside the range of 16°C to 40°C (61°F to 104°F), 15-95% RH non-condensing>. Do not use if the device malfunctions or has been damaged in any manner.

There is generally no restricted “shelf-life” of the device.

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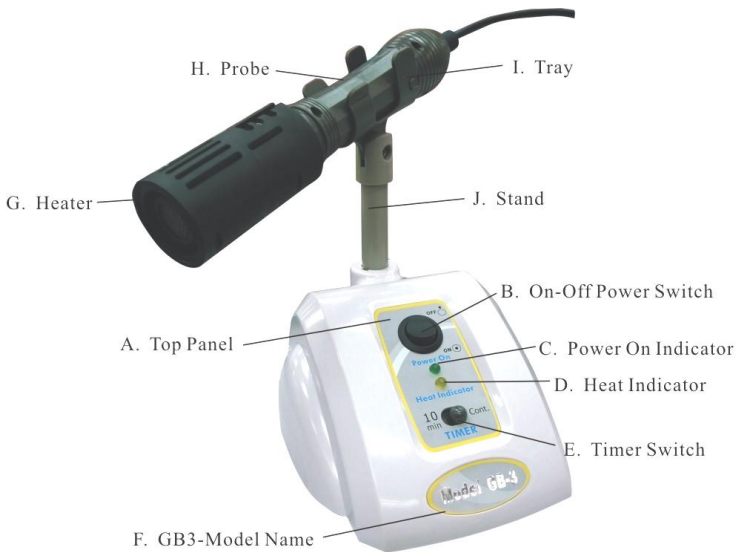
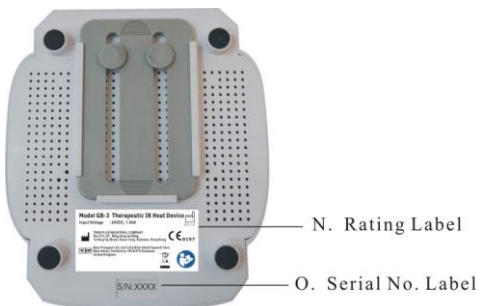
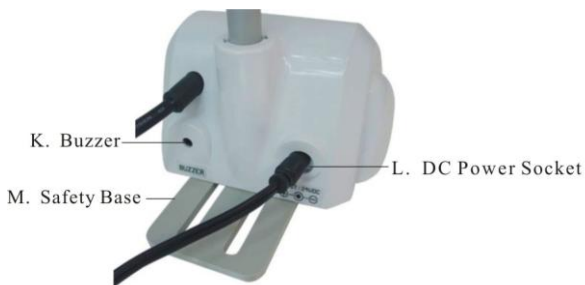
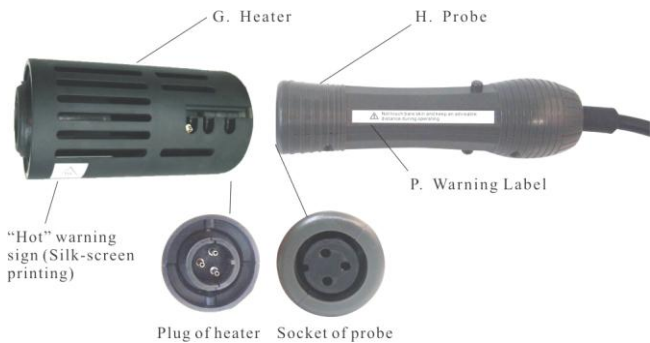
Part No. : WR9QI1040CCP-N-IM

Electrical Specifications :

INPUT VOLTAGE:	100-240 VAC
INPUT CURRENT:	< 0.6 A RMS MAX
INPUT FREQUENCY:	47 - 63 HZ
OUTPUT VOLTAGE :	24 VDC
OUTPUT CURRENT :	1.04A, NO MINIMUM LOAD REQUIRED
OUTPUT POWER (RATED):	25 WATTS MAXIMUM
OUTPUT RIPPLE VOLTAGE (PEAK TO PEAK):	+/-1% OR 50 mV WHICHEVER IS GREATER AT NOMINAL OUTPUT VOLTAGE. MEASURED AT 20 MHz BANDWIDTH WITH 0.1µf CERAMIC CAPACITOR IN PARALLEL WITH 10 µf ELECTROLYTIC CAPACITOR CONNECTED AT THE END OF THE OUTPUT CONNECTOR.
OUTPUT REGULATION:	± 5 % OF NOMINAL OUTPUT VOLTAGE, MEASURED AT O/P CONNECTOR AT NOMINAL LINE
LINE REGULATION:	± 1 % MAX AT FULL LOAD
TURN ON / TURN OFF OVERSHOOT:	5% MAX 500µS MAX RECOVERY TIME FOR 25% TO 50% STEP LOAD
TURN-ON DELAY:	3000 mSEC MAX AT FULL LOAD, 115 VAC
HOLD-UP TIME:	10 mSEC MIN AT NOMINAL INPUT AND FULL LOAD
INRUSH CURRENT:	30 A MAX AT COLD START 120 VAC, 60A MAX COLD START 230VAC
SWITCHING FREQUENCY:	66.5 KHz TYPICAL
OVER-VOLTAGE PROTECTION:	PROTECTED, HICCUP WITH AUTO RECOVERY
OVER-LOAD:	PROTECTED, HICCUP WITH AUTO RECOVERY
SHORT CIRCUIT:	PROTECTED, HICCUP WITH AUTO RECOVERY
NO LOAD POWER CONSUMPTION:	MEETS ENERGY STAR LEVEL IV 115 VAC
OUTPUT INDICATOR:	BLUE LED FOR OUTPUT PRESENT
OTHER PROTECTION:	INPUT LINE AND NEUTRAL FUSING
SAFETY APPROVALS :	UL60601-1, CUL TO 22.2 # 601, DEMKO TO EN60601-1, UL60950-1, CUL TO 22.2 # 950, DEMKO TO EN60950-1 CB REPORT TO IEC 60601-1:2005 (3rd EDITION), CE, 5656 VDC PRIMARY-SECONDARY
DIELECTRIC WITHSTAND VOLTAGE:	≤ 0.3 mA AT 240 VAC
LEAKAGE CURRENT:	COMPLIES WITH EN55011 AND EN55022 CLASS B, VCCI AND FCC PART 15 CLASS B WHEN TESTED WITH A RESISTIVE LOAD, BOTH CONDUCTED AND RADIATED EMI
EMI:	TESTED TO COMPLY WITH EN 60601-1-2:2001, EN61000-3-2 , EN61000-3-3, INCLUDING EN61000-4-2, EN61000-4-3, EN61000-4-4, EN61000-4-5, EN61000-4-6, EN61000-4-8, EN61000-4-11
CE MARK:	COMPLIES TO SECTION 301 OF ENERGY INDEPENDENCE AND SECURITY ACT (EISA) COMPLIES WITH ENERGY STAR TIER 2 (NORTH AMERICA), CECP TIER 2 (CHINA), MEPS TIER 2 (AUSTRALIA), CODE OF CONDUCT (EUROPE) 5.0 ~ 6.9V WITH 1000 mm CABLE ONLY
EFFICIENCY:	200,000 HOURS AT 25°C AMBIENT TEMPERATURE
MTBF:	0°C TO 40° C
OPERATING TEMPERATURE:	-40°C TO 80° C
STORAGE TEMPERATURE:	0% TO 90% RELATIVE HUMIDITY
HUMIDITY:	

Explicación símbolos

The following symbols are used in this device.

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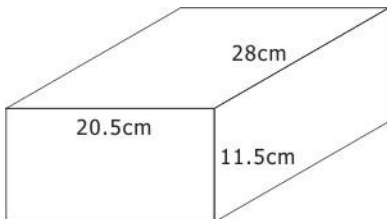
Table 201 –Declaration - electromagnetic emissions

Guidance and manufacturer's declaration - electromagnetic emissions		
The TP77 is intended for use in the electromagnetic environment specified below. The customer or the user of the TP77 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 2	The TP77 must emit electromagnetic energy in order to perform its Intended function. Nearby electronic equipment may be affected.
RF emissions CISPR11	Class B	The TP77 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

Table 202 - Declaration - electromagnetic immunity -

Guidance and manufacturer's declaration — electromagnetic immunity			
The TP77 is intended for use in the electromagnetic environment specified below. The customer or the user of the TP77 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Table 204 - Declaration - electromagnetic immunity -

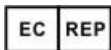
Guidance and- manufacturer's declaration. Electromagnetic immunity			
The TP77 is intended for use in the electromagnetic environment specified below. The customer or the user of the TP77 should assure that it is used in such an environment.			
Immunity test	IEC 60501 test level	Compliance level	Electromagnetic environment - guidance
			<p>Portable and mobile RF communications equipment should be used no closer to any part of the TP77 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p>
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	$d = 12\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = 12\sqrt{P}$, 80MHz to 800MHz $d = 23\sqrt{P}$, 800MHz to 2,5MHz
			<p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<p>^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Model Number or Type Number - by manufacturer is used exceeds the applicable RF compliance level above, the Model Number or Type Number - by manufacturer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Model Number or Type Number - by manufacturer.</p>			
<p>^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than fV_j V/m.</p>			

Table 206 - Recommended separation distances between portable and mobile RF communications equipment and the TP77

Recommended separation distances between portable and mobile RF communications equipment and the TP77			
The TP77 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the TP77 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the TP77 as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 12\sqrt{P}$	80 MHz to 800 MHz $d = 12\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0.12	0.12	0.23
0,1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) accordable to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

Tens Plus Industrial Company
Room C12, 3/F., Wing Hing Ind. Bldg.
14 Hing Yip St., Kwun Tong
Hong Kong

Blair Prospect Limited
Blair Atholl, Perthshire,
PH18 5TL Scotland, Unit Kingdom

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